



# THUASNE

## **BIPLAST®**

*Bande adhésive*

*Adhesive bandage*

*Klebebinde*

*Kleefverband*

*Benda adesiva*

*Venda adhesiva*

*Ligadura adhesiva*

接着テープ

*Klebebind*

*Klæbende bind*

*Liimaside*

*Häftband*

*Κολλητική λωρίδα*

*Obinadlo s lepicími  
pásky*

*Bandaż elastyczny  
samoprzylepny*



**THUASNE**

BP 243 - 92307 Levallois-Perret Cedex - France

Commandes / Orders (St-Etienne) : Tél. 04 77 81 40 42 (France)

Tel. +33 (0)4 77 81 40 01/02 (Export)

**Thuasne Deutschland** - Burgwald

Tel. (06451) 7212-0 - Fax (06451) 7212-50

**Thuasne Belgilux** - Mechelen

Tel. (0)15 27 71 27 - Fax (0)15 27 21 51

**Thuasne Nederland** - Nijkerk

Tel. 033 - 247 44 44 - Fax 033 - 247 44 43

**Thuasne Italia** - Desio

Tel. 0362 33 11 39 - Fax 0362 30 79 17

**Thuasne España** - Madrid

Tel. 91 519 87 82 - Fax 91 415 03 51

**Thuasne Czech Republic** - Praha

Tel. 02/5718 9851-2 - Fax 02/5721 6521

# BANDE ADHESIVE

(F)

## Propriétés :

- Bande adhésive de contention.
- Tissu élastique aéré.
- Enduction multi-directionnelle en chevrons.
- Bonne tolérance cutanée.
- Film protecteur intercalaire.

## Indications :

- Traumatologie : contention articulaire (entorse, luxation, tendinite) ou musculaire.
- Phlébologie : réduction et stabilisation des œdèmes.

## Précautions :

Suivre les conseils du professionnel qui a prescrit ou délivré le produit.

- En cas de gêne importante ou de sensations anormales, ôter le produit et demander conseil à ce professionnel.
- Ne pas appliquer directement sur une plaie.
- Respecter la méthode de pose et la durée indiquées par votre médecin.
- Intercaler de préférence une bande-mousse de protection, surtout pour les peaux sensibles.

Stocker à l'abri de la chaleur.

Produit à usage unique.

*Conserver cette notice.*

# ADHESIVE BANDAGE

(ENG)

## Properties :

- Elastic adhesive bandage.
- Aerated elastic fabric.
- Multidirectional chevron patterned adhesive backing.
- Good skin tolerance.
- Protective, release backing paper.

## Indications :

- Traumatology : joint or muscle compression (sprain, dislocation, tendinitis).
- Phlebology : reduction and stabilization of oedema.

## Precautions :

Follow the advice of the professional who prescribed or supplied the product.

- In the event of serious discomfort or abnormal sensations, remove the product and consult this professional for advice.
- Do not apply directly to an open wound.
- Follow your doctor's instructions concerning the fitting method and how long the bandage is to be worn.
- Preferably insert a protective underwrap between the skin and the adhesive bandage, especially if you have sensitive skin.

Store in a cool place.

Disposable product to be used once only.

*Keep this instruction leaflet.*