THUASNE

LOMBASKIN®

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Composition :

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Conserver cette notice.

“SECOND SKIN” COMPACT LUMBAR SUPPORT BELT

Belt specially designed for lightness and anatomical fit.

Properties :

- Lightweight fabrics used: 10% lighter than a conventional belt.
- Aerated elastic Combitex® fabric (patented).
- Four back stays fitting anatomically.
- Two flexible front stays.

Indications :

- Individuals suffering from backache.
- Acute, sub-acute or chronic lumbar pain with or without nerve pain.
- Post-operative lumbar and pelvic support,
- Individuals with a connective tissue disease (e.g.: Ehlers-Danlos syndrome).
- Individuals with sensitive skin.
- Fibromyalgia.

Fitting instructions :

- **ADJUST THE STAYS TO THE SHAPE OF THE BACK BEFORE USING FOR THE FIRST TIME.**
- Put on the belt, with the hand loop on the right (see diagram on page 2).
- Put on and fasten the belt using the hand loop and thumb loop.

Precautions :

Follow the advice of the professional who prescribed or supplied the product.

Re-use of this device for the treatment of another patient is strictly not recommended due to hygiene- and performance-related problems. If a decision is taken to re-use this device, then the person taking the decision is solely responsible and the Thuasne warranty will automatically become void.

Washing instructions:

- Wash only with soapy water, without detergent or bleach.
- If machine-washing, ensure that the Velcro fasteners are closed by placing them on top of one another.
- Dry flat away from heat sources.

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