# THUASNE

---

## **LOMBACROSS ACTIVITY®**

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## Composition :

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# POSTURE-CORRECTING REINFORCED LUMBAR BELT

(ENG)

## Properties :

- Anatomical cut for greater comfort, particularly for women.
- Aerated elastic Combitec 2® fabric (patented).
- Optimized back comfort due to a fleecy-foam complex.
- Crossed-over straps of fabric (muscular function analogy). (1)
- Rigid, anatomically-fitting back stays. (2)
- Additional straps for adjusting the level of back support. (3)
- Hand loop to make it easier to put the product on.

## Indications :

- Acute and sub-acute lumbar pain and sciatica.
- Lumbar support following surgery or immobilisation.

## Fitting instructions :

- Unfasten the lateral Velcro straps.
- Put on the belt, with the inside label upwards.
- Using the hand loop, fasten the ventral Velcro strap, adjusting it to the desired level of support.
- Adjust dorsal support by fastening the additional lateral straps at the front using the finger loops.

## Precautions :

Follow the advice of the professional who prescribed or supplied the product. It may be necessary to reshape the stays. In the event of discomfort, consult your professional if discomfort occurs.

Store at room temperature, preferably in its original packaging.

Re-use of this device for the treatment of another patient is strictly not recommended due to hygiene- and performance-related problems. If a decision is taken to re-use this device, then the person taking the decision is solely responsible and the Thuasne warranty will automatically become void.

## Washing instructions :

- Wash only with soapy water, without detergent or bleach.
- Squeeze out excess water.
- Dry away from heat sources.

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