es


ca

en

FR

DE

IT

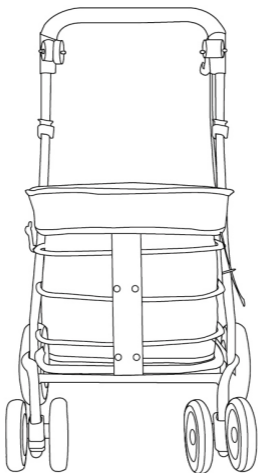# CARLETT

easy shopping

Lett**700**

español

# Instrucciones

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# Instructions

en

Thank you for purchasing this Carlett product. Remember to keep your purchase receipt as it is the only way to validate the product guarantee in the event of any defects.

Any defects that arise from improper use by the end user, as well as due to any normal wear and tear of the product's components (wheels and textile components) are not included in the guarantee.

We advise you to carefully read the instructions before using the trolley in order to use it correctly.

We hope you enjoy using this product, which has been designed to help improve the quality of life of its users.

Thank you!

## Initial installation

- 1) Insert the handle into the chassis and alter the height to the available positions.
- 2) The trolley is ready to be used.

## Lett700, practical and comfortable

A great sized trolley to help you with all your shopping needs. Do your shopping while seated. An ideal trolley for running errands along with the option to sit down and take a break for a few minutes and start up again with renewed energy. A product designed for a wide age group, for independent people that go out daily and are looking for a practical and handy assistant that will carry their personal possessions (wallet or

purse, keys, mobile phone, etc.), as well as being able to easily transport shopping.

It comes with a **safety or parking brake**, essential when using the trolley as a seat. To activate the safety



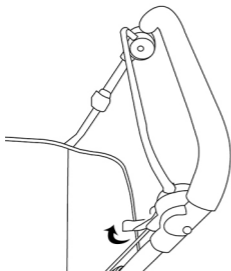
Diagram 1. Safety brake

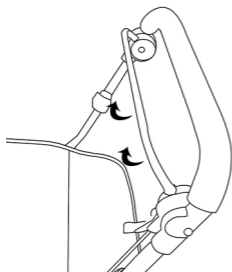Esquema 2. Instant braking

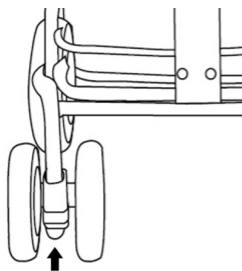
Esquema 3. Rotating wheels

brake, pull the brake handle backwards. To remove the brake and to move forward again, push the handle forwards (see diagram no. 1).


It comes with an **instant braking system** that is activated by pulling on it until the trolley stops. To remove the brake and to move forward again, simply release the brake handle (see diagram no. 2).

The trolley also comes with a **front wheel rotation level adjustment system**, which can be used according to your needs. The wheels may be completely fixed or can rotate freely (360°). To change the rotation level, press on the adjustment mechanism located on the front wheels and adjust as needed (see diagram no. 3).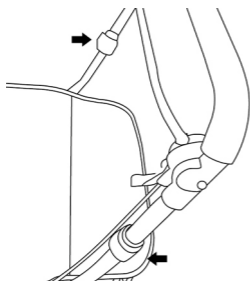

Esquema 4. Adjustable handlebar

It comes with a **three position handle height adjustment mechanism**. To change height, remove the locks from the holes located on the right and left side of the handle, set to the required height and fix again by inserting the locks in the corresponding holes on both sides (see diagram no. 4).

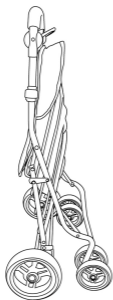Esquema 5. Folded system

To fold and store the trolley, **simply pull the seat cushion upwards** (see diagram no. 5).

which is easy to clean.

It comes with a removable interior bag with handles,

## Materials

- Chassis: aluminium
- Textile components: polyester
- Wheels: EVA material (this material allows the wheels to run silently even over rough ground)
- Wheels: polypropylene

## Specifications

- Unfolded product: W 49 x L 50 x H 84-91 cm.
- Folded product: W 49 x L 27 x H 84-83 cm.
- Seat measurements: W 35 x L 28 cm
- Maximum seat load: max. 100 kg
- Maximum load of removable interior bag: max. 15 kg
- Removable bag capacity: 32 liters
- Net weight of chassis: 5.2 kg

## Maintenance

For daily cleaning please use a dry cloth. If

you need to remove any stains, we advise you to use a neutral detergent and clean the stain by hand.

In order to lengthen the life of the product, we advise you to dry the chassis with a dry cloth if the trolley comes in contact with rain or water.

Do not modify or disassemble the product as this may cause malfunction or may make the trolley unsafe for use.

## Recommendations

This product is **only a seated shopping trolley and is aimed at allowing the transport of shopping items.** The manufacturer shall not be liable should the product be used for any other use for which it is not intended.

Do not move the trolley when the seat is in use.

The **maximum load weight for the bag is 15 kilos.** Do not force the bag even when the maximum load weight is not reached. Always place the heaviest items at the bottom of the bag. Pay special attention when using the product on a ramp or incline, especially when loaded.

Remember to use the instant handbrake to control the trolley.

Pay special attention when using the trolley on wet or slippery surfaces.

Before using the trolley as a seat, check the safety or parking brake is on.

This product is not a toy. Do not allow this trolley to be used by children or allow children to climb on it.

Do not use the interior of the trolley to transport domestic animals or pets.

Do not store the trolley in damp or very hot places, or indirect sunlight, as this may shorten its useful life.

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