# Homecraft Bed Grab Rail

NEW CODE	OLD CODE
09 108 9481	AA3475


## User Instructions

Please read these instructions fully before assembling and fitting the Bed Grab Rail to the bed.

## Item Description

An ultra sturdy bed grab rail, offering stability and confidence to those that have difficulty getting into and out of bed. The height adjustable frame can be positioned to suit virtually any divan style bed. Two legs rest upon the floor and the frame is secured into place via the three colour coded webbing straps provided, ensuring minimal movement when in use.

## To assemble the Bed Grab Rail

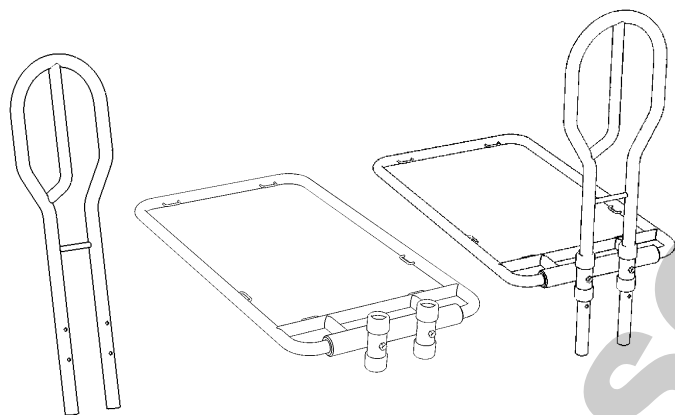First determine the top side of the base frame by laying the short coloured lengths of webbing outwards (already fitted to the frame base). The adjustment buckles should be uppermost. The handle tube has four holes in one side. Remove the grey sleeves from the end of the handle if necessary and push the legs of the handle into tubes on the front of the base frame ensuring that the holes are facing away from the base frame. Replace the two grey sleeves onto the ends of the handle tubes.

## Adjusting the Bed Grab Rail to suit the bed

The Bed Grab Rail has the option of being fitted with legs or not. It is recommended that the product is used with the legs fitted for maximum support. The legs are suitable for use with all standard height beds. Do not use the legs if the Bed Grab Rail is to be fitted to a high bed or one that has been raised up.

## Assembling the Bed Grab Rail with the legs

To fit the legs, slide each leg into the handle with the holes to the front. Remove the mattress from the bed and place the Bed Grab Rail on the bed base. Align the holes in the leg with the holes in the handle so that the rubber feet are either touching or are just above the floor. Push the pin clips through the aligned holes to secure the legs in place. Push down on the top of the handle until the rubber feet touch the floor. Screw the tri-cornered thumb screws into the threaded holes on the front of the base frame fixing tubes to clamp the handle in place.

## Assembling the Bed Grab Rail without the legs

If the legs are not long enough to suit the bed the handle can be locked in place to give the necessary support. With the legs removed push the pin clips into the redundant lower holes in the handle. With the top edge of the pin clips positioned just below the fitting tubes on the base frame, a second set of holes in the handle tube will be aligned with the threaded holes in the fixing tubes. Look through the threaded holes to ensure the holes are aligned and then screw the tri-cornered thumb screws into the threaded holes on the front of the base frame fixing tubes. The screws locate all the way in, until the head of the screws are touching the fixing tubes. If they do not then the holes were not aligned and the screws should be loosened and handle tube repositioned as necessary.

## Fitting the Bed Grab Rail to the bed

If not already done, remove the mattress from the bed and place the Bed Grab Rail on the bed base.

Position the Bed Grab Rail where it is required. The Bed Grab Rail is supplied with a full set of webbing straps to fully secure the product to the bed base. There are two shorter green straps and one longer blue strap. These should be passed around the width and length of the bed respectively and then clipped into the corresponding coloured buckles on the short lengths of webbing that are already fitted to the base frame. The straps are purposefully long to assist in passing them under the bed. When the straps have been clipped into place the free ends of the straps can be pulled to tighten the straps. Adjust the position of the Bed Grab Rail if required, check the Bed Grab Rail is fully secured and replace the mattress.

Please retain these instructions for future reference.

Maximum user weight  

**Authorised Distributed & Representative: Homecraft Rolyan**, A Patterson Medical Company, Nunn Brook Road, Huthwaite, Sutton-in-Ashfield, Nottinghamshire, UK, NG17 2HU

**UK:** Tel: 01623 448 702 Fax: 01623 557 769 www.homecraft-rolyan.com

**International:** Tel: +44 1623 448 706 Fax: +44 1623 448 784

**Sammons Preston Rolyan**, 4 Sammons Court, Bolingbrook, IL 60440-5071, USA.

Tel: 1-630-226-0000 Telefax: 1-630-226-1390

**homecraft**  
**Rolyan**  
A Patterson Medical Company

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