THUASNE

LOMBAX®

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Reinforced lumbar support belt

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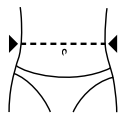
ENG

Properties :

- Strong compression elastic fabric.
- Lumbar reinforcement with 4 stays. (1)
- Abdominal reinforcement with 2 stays. (2)
- Two-part fastening system. (3)
- Easier to hold and fasten due to several hand loops. (4)
- Improved comfort due to 3D knit.

Indications :

- Initial support of acute or chronic lumbar pain and sciatica.
- Prevention of lumbar pain (strain, incorrect posture).
- Post-operative lumbar support.

Fitting instructions :

- First, slip your right hand through the lower strap with the label marked "THUASNE". Place the belt around your waist (inside label upwards). Hold the belt with your left hand flat against your stomach, without pulling.
- Pull with your right hand (in front of you then towards the left) and fasten the lower strap on the opposite lower part.
- Repeat this operation with the upper strap.
- Check with your hand that the fabric between the two central stays at the back (there are 4 in total at the back) is properly positioned against your spine to make sure the belt is centred as well as possible.

For improved comfort, loosen the upper strap when sitting down.

Precautions :

Follow the advice of the professional who prescribed or supplied the product. It may be necessary to reshape the stays. Consult your professional if discomfort occurs.

Store at room temperature, preferably in its original packaging.

Re-use of this device for the treatment of another patient is strictly not recommended due to hygiene- and performance-related problems. If a decision is taken to re-use this device, then the person taking the decision is solely responsible and the Thuasne warranty will automatically become void.

Washing instructions :

- Wash only with soapy water, without detergent or bleach.
- Squeeze out excess water.
- Dry away from heat sources.

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