



## **Viscount Raised Toilet Seat with Arms**

**VR224J**



## **Usage and Maintenance Instructions**

## INTRODUCTION

Thank you for purchasing the Viscount Raised Toilet Seat with Arms from Aidapt.

## BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

## INTENDED USE

The Viscount Raised Toilet Seat with arms is designed to fit the majority of UK toilet bowl shapes. The arms are provided to assist the user when getting on and off the seat. Please note the user must not use arms to support their full body weight as this could result in damaging the unit.

## SAFETY

- Consult with your physician or supplier to determine proper adjustment and use of the device.
- Do not lean out of the Viscount Raised Toilet Seat to avoid falling out of the device.
- The Viscount Raised Toilet Seat is intended for indoor use only.
- Carefully check all screws are tightened before use.
- Review weight limit and weight load before use. Do not use exceed the weight limit.
- Do not make any modifications on this device.
- Always check stability before use.

## CLEANING

Clean your Viscount Raised Toilet Seat using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners may damage the product beyond repair and should not be used. Always ensure your toilet seat is wiped dry after cleaning.

If to be disinfected by heat, one of the following three temperatures and duration of exposition can be used:

- a) a temperature of 90°C for 1 minute
- b) a temperature of 85°C for 3 minutes
- c) a temperature of 80°C for 10 minutes

## INSTALLATION

- 1 – Ensure you inspect the product for any visual damage before use.
- 2 – Carefully remove the existing toilet seat and lid from the toilet.
- 3- To install arms, you must depress the push pins on the arms and insert them into the mounting tubes. Ensure they are locked in place. To remove the arms, reverse this step.



*fig. 1*

- 4 – Carefully place the seat onto the toilet bowl. Position the raised toilet seat correctly on the rim of the toilet ensuring the front lip is outside the bowl. (see fig. 1).
- 5- Ensure the front adjustment bracket clears the rim so that the toilet seat fits securely.



fig. 2

6- To tighten, turn the hand wheel clockwise (fig. 2).

7- Check the stability before use.

8 – To remove, simply unscrew the hand wheel and remove the raised toilet seat by lifting up.

#### **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

#### **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

#### **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.